

# Caring for your Eye Fashion Accessories

Please ensure that you read all the instructions in this leaflet before attempting to first wear your new EYE 2 EYE Contactz™ Colour Eye Accessories.

By following these instructions you will get the most from your new look.

## Preparing to wear

- Hygiene is an essential part of preparation.
- Always wash and dry your hands thoroughly, removing traces of anything that may cause contamination or damage, before handling.

## Achieving your look

- Remove from packaging
- Try to make a habit of inserting the right side first and always make a distinction between right and left.
- Place on your index finger and check that your colour eye accessory is clean and not inside out, ensuring that it has a natural bowl like shape.
- Hold your upper lid open with your left hand's index and middle finger, at the same time hold your lower lid open with the middle finger of your right hand. Looking straight at your eye in the mirror, slowly and gently place the accessory on your eye.
- Carefully release your eyelids and blink.
- You can gently massage your eyelid to centre the accessory.
- When you are sure that it is correctly inserted, repeat the procedure for the left eye.

## Removing

- Wash and dry your hands thoroughly.
- Look upwards and hold down the lower lid with the middle finger of your right hand. Placing your index finger and thumb of the same hand on opposite sides of the accessory, slowly slide it down to the white of your eye.
- Gently pinch the accessory from your eye, taking care not to pinch too hard.
- Repeat for the other eye.

## Wearing Instructions

- EYE 2 EYE Contactz™ Colour Eye Accessories are intended for occasional use only. Check with your eyecare practitioner before wearing for prolonged periods on an everyday basis. Your eyecare practitioner will advise you of your recommended wearing schedule.
- Always rinse your accessories with EYE 2 EYE Contactz™ All-In-One solution before wearing and after removing.
- After each wear, always store your colour eye accessories in fresh EYE 2 EYE Contactz™ All-In-One solution in the case provided.
- If you wear your colour eye accessories infrequently, always remember to replace the solution used for fresh solution regularly.
- EYE 2 EYE Contactz™ Colour Eye Accessories are for monthly use only. They are intended to be replaced 30 days after opening, irrespective of the number of times worn during such 30 day period.

## Safety

- **DO NOT** wear if your colour eye accessories appear to be damaged.
- **DO NOT** share your colour eye accessories with anyone else.
- **DO NOT** let hairspray or any other cosmetic aerosols come into contact with your EYE 2 EYE Contactz™.
- **DO NOT** participate in any water sports whilst wearing.
- **ALWAYS** remove your colour eye accessories before you go to bed.
- **ALWAYS** put your eye accessories in before applying any make-up.
- **ALWAYS** take out your eye accessories before removing any make-up.
- **ALWAYS** remove immediately if your eyes get red or irritated, or if you have any discomfort or pain, and seek help from a qualified eyecare practitioner.

## Important Information

- Please note that EYE 2 EYE Contactz™ Colour Eye Accessories are not intended to correct any defect of sight and are intended only for cosmetic colour change.
- If you suffer from any of the following, then you **MUST NOT** wear EYE 2 EYE Contactz™:
  - Corneal dystrophies
  - Corneal ulcers
  - Inflammation of the cornea, e.g. keratitis
  - Inflammation of the eye, e.g. iritis, uveitis
  - Allergic eye conditions
  - Any pre-existing eye condition

If you are in any doubt, please seek further advice from your medical or eyecare practitioner

- EYE 2 EYE Contactz™ Colour Eye Accessories are suitable for normal, healthy eyes (size: 14.5mm diameter, median base curve). If you are in any doubt as to whether these eye accessories are suitable for you, then please consult a medical or eyecare practitioner.

## Fitting and Aftercare

- It is recommended that you consult your eyecare practitioner to ensure suitability.
- Always have regular check-ups and follow the advice of your medical or eye practitioner.
- If you feel any unexplained discomfort, watering, vision change or redness, remove immediately and seek help from a qualified eyecare practitioner.

If you have any questions or concerns regarding the information in this leaflet, please consult your medical or eyecare practitioner.



## How do I choose the right colour for me ?

EYE 2 EYE Contactz™ offers a wide palette of beautiful colours, in a variety of tones. Colour selection depends on your natural eye colour, and desired look.

## Will EYE 2 EYE Contactz™ colour eye accessories really change the colour of my eyes ?

Yes. EYE 2 EYE Contactz™ wearers report great satisfaction with their instant eye colour change, no matter what their original eye colour. Your look can be changed quickly and effectively to suit any mood, outfit, occasion...

## Are they easy to wear ?

Yes. Regular and new wearers do find them easy to wear and extremely comfortable, which is why first time users keep coming back for new colours and new looks.

## Are they affordable ?

The EYE 2 EYE Contactz™ brand is priced very competitively, offering you excellent value for money over other similar products. See your local stockist for the latest offers and colour options.

## Why EYE 2 EYE Contactz™ ?

Outstanding quality, value for money in addition to the varied choice of vivid colours and new look options, make EYE 2 EYE Contactz™ ...

...THE LEADING CHOICE IN EYE FASHION TODAY

## OTHER GREAT PRODUCTS AVAILABLE FROM EYE 2 EYE Contactz™

**EYE 2 EYE Contactz™ All-in-One**  
Cleans, Rinses, Disinfects, Wets, Stores, Lubricates & Removes Protein  
Keeps Lens Moist and Enhances Wearing Comfort  
*Includes Free Lens Case*

**EYE 2 EYE Contactz™ Hydro**  
Cleans, Rinses, Disinfects, Wets, Stores, Lubricates & Removes Protein  
Specially Formulated for Silicon Hydrogel Lenses  
*Includes Free Lens Case*

**GLAMOUREYEz™**  
Gently Soothes Redness and Brightens.  
Notice the Difference in the Blink of an Eye

**COMFORT DROPz™**  
Refreshes and Moisturises Soft Lenses  
For All Day Wearing Comfort

**EYE WASH by EYE 2 EYE Contactz™**  
Cleans, Freshens & Soothes  
*Includes Eye Bath*

For further advice you can call our helpline on

**0870 2000 EYE**  
(0870 2000 393)

## ACCESSORIZE WITH COLOURED EYES

Your local EYE 2 EYE Contactz™ stockist is:

[www.eyeyecontactz.co.uk](http://www.eyeyecontactz.co.uk)

Visit our website for a unique interactive experience and further information

Colours shown are for illustrative purposes only. These eye accessories are for cosmetic use only, are not designed to correct any defect of sight and are governed by European cosmetic product regulations. Recommended care instructions should be followed at all times.



# ACCESSORIZE WITH COLOURED EYES



*seeing is believing*

**AS SEEN ON TV!**

WORN BY THE CAST OF THE HIT

**SKY000**

SHOW "DREAM TEAM"

"Dramatic Colours"

"Comfortable and Easy to Use"

**EYE 2 EYE Contactz™**

# NEW COLOURS

## BASIC

A single tone accessory suitable for a basic quick new colour look.



## MIXED

A mixed tone accessory specially created for dark eyes to deliver a dramatic new look.



# NEW LOOK


## ENHANCED

A two tone accessory with a dark outer edge to give a wide-eyed enhanced new colour look.



## BLACK


A black tone accessory designed to create a completely unique customised look.

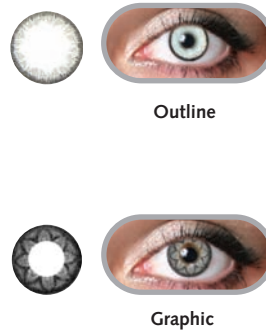

# NEW YOU

## NATURAL

A three tone accessory designed to blend for the most natural new colour look.

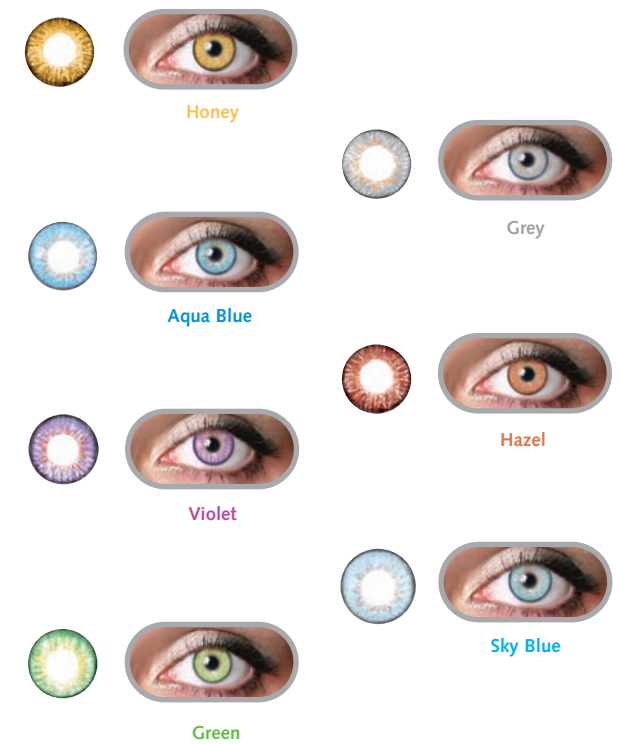

The Right Colours for The Right Look at The Right Price  
Always Insist on **EYE 2 EYE Contactz™**

**...THE LEADING CHOICE IN EYE FASHION TODAY**